

Dispogel[®]

**Urethral Lubricating Gel
with 2% Lidocaine HCL**

Dispogel is a sterile lubricant gel containing local anaesthetic (Lidocaine 2% and antiseptic (Chlorhexidine Digluconate 0.05%) Dispogel is available in 11ml single use pre-filled syringes each presented in a sterile package.



- ✓ Dispogel is used for insertion into the urethra prior to insertion of a catheter or other medical instruments. The insertion of urinary catheters together with Dispogel Urethral Lubricant Gel with Lidocaine HCL (11ml) health professionals minimise associated trauma, pain and catheter associated infections.
- ✓ After insertion of gel, wait for 5-10 minutes prior to insertion of a catheter or other medical instruments – time needed for achieving anaesthetic effect.

Sterile Lubricant
with
Antiseptic
with
Anaesthetic

11ml STERIL R CE2292

PRODUCT CODE	PRODUCT DESCRIPTION	BRAND	PACK SIZE
DSP3	11 ml Syringe	Dispogel	Pack of 10



Engineers and Doctors (Pty) Ltd
**ENGINEERS
& DOCTORS**

Engineers and Doctors (Pty) Ltd

T: +27 12 880 4300

E: orders@endoc.co.za